



BY ACCESSIBILITY PROFESSIONALS

FREEDOM JOURNEY LIFTS

QUICK START INSTALLATION - MANUAL #5



Customer Support: 1.877.947.7769
Email: apsupport@accessibilitypro.com

Instructions


For details see Residential VPL Installation Manual.




1) Remove packaging. The lift is shipped with the platform in an upright position, as shown. Attach the base legs to the tower. Install switches in a marked space.



2) Lower the platform as shown and tighten the platform mounting bolts (1/2" bolts). Check that the emergency stop button is not depressed and that the on/off key is in and turned clockwise. Plug the lift into the wall outlet (standard 110V 15A outlet). Test the lift operation by pressing the up and down paddles.

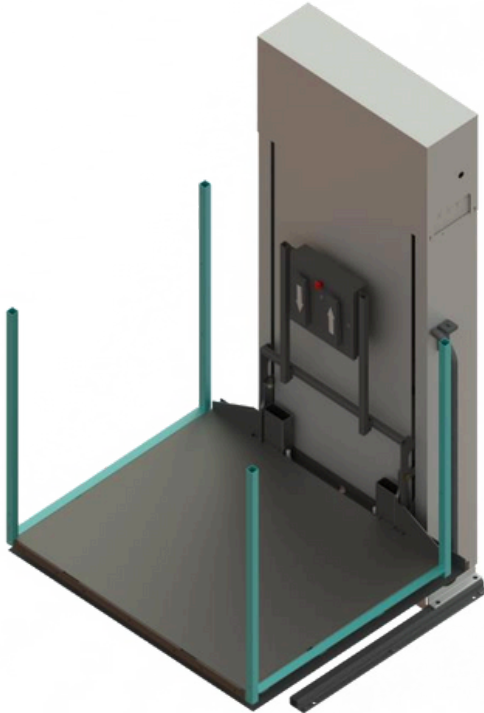

3) Attach 1 1/2" extension weldments to the lift platform using 5/16" hex head screws, washers and lock nuts.

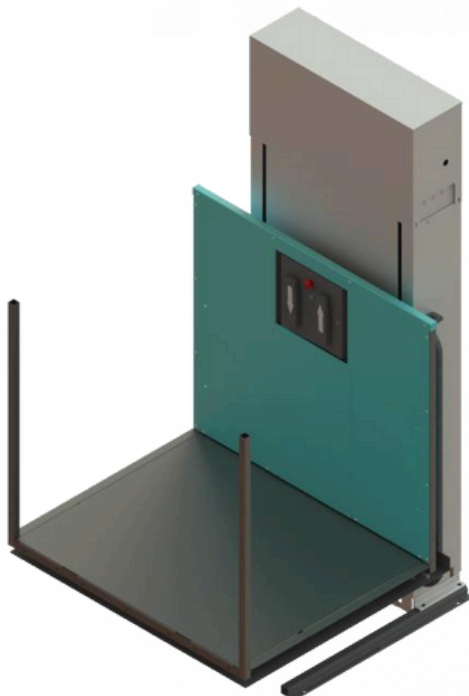

4) Attach the handrail top and bottom tubes to the platform handrail control wall using 3/16" rivets. Add plugs to the top handrail tube. Attach the platform handrail control wall to 1 1/2" extension weldment posts using 3/16" rivets. Attach the inspection/maintenance panel to the platform handrail control panel using #10-24 screws.



5) Attach the handrail top and bottom tubes to the platform handrail solid wall using 3/16" rivets. Attach the platform handrail solid wall to 1 1/2" extension weldment posts using 3/16" rivets. Add plugs to the top handrail tube.



6) Attach safety pan extensions to the platform main safety pan using brackets and 1/8" rivets. See lift assembly drawing for parts and details.



7) Attach the toe plate assembly to the lift using 3/16" rivets.

Lift assembly shown for reference is 36" x 48" with right side toe plate. Procedure is the same for all configurations.